

**I. MULTIPLE CHOICE**

- What is the length of time required for specimen collection of oral fluids?
  - 1 minute
  - 2 minutes
  - 3 minutes
  - 4 minutes
- The absence of a line next to a drug abbreviation indicates a \_\_\_\_\_ for that particular drug.
  - Non negative
  - Negative
  - Invalid
  - All of the above
- How long should you wait after a person has eaten or drank before performing the oral specimen collection?
  - 10 minutes
  - 15 minutes
  - 1 minute
  - 20 minutes
- If the result is non negative, how do you secure the specimen for laboratory confirmation?
  - Place sponge in a clear plastic bag and send to a lab.
  - Secure cap with security seal and package the device for transport to a lab.
  - Place cap in a clear plastic bag and send to a lab.
  - None of the above.

**ANSWER:** \_\_\_\_\_

**ANSWER:** \_\_\_\_\_

**ANSWER:** \_\_\_\_\_

**ANSWER:** \_\_\_\_\_

**II. MATCHING** (Match the appropriate abbreviation to the drug name)

- |                     |       |      |
|---------------------|-------|------|
| 1. Marijuana        | _____ | COC  |
| 2. Amphetamines     | _____ | PCP  |
| 3. Cocaine          | _____ | mAMP |
| 4. Opiates/Morphine | _____ | AMP  |
| 5. Phencyclidine    | _____ | THC  |
| 6. Methamphetamine  | _____ | OPI  |

**Please fill out your contact information to receive a Certificate of Completion:**



Name \_\_\_\_\_

Company \_\_\_\_\_



Address \_\_\_\_\_

Phone (\_\_\_\_) \_\_\_\_\_


**III. IDENTIFY THE RESULTS AS EITHER: NON NEGATIVE, NEGATIVE, OR INVALID**

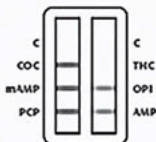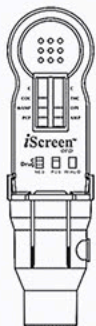

\_\_\_\_\_

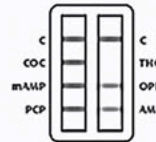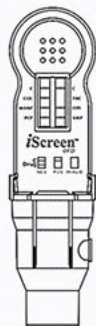

\_\_\_\_\_

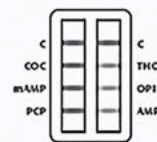

\_\_\_\_\_