

### I. MULTIPLE CHOICE

1. When collecting specimen in a cup, what should the temperature range be?
  - a. 90-100° Celsius
  - b. 85-95° Fahrenheit
  - c. 90-100° Fahrenheit
  - d. 85-95° Celsius

**ANSWER:** \_\_\_\_\_

2. The absence of a line next to a T indicates a \_\_\_\_\_ for that particular drug.
  - a. Non negative
  - b. Negative
  - c. Invalid
  - d. All of the above

**ANSWER:** \_\_\_\_\_

3. When administering the test, immerse the test strips vertically in the urine sample for at least \_\_\_\_\_.
  - a. 30-45 seconds
  - b. 1 minute
  - c. 10-15 seconds
  - d. None of the above

**ANSWER:** \_\_\_\_\_

4. Negative results may be interpreted as soon as all lines are visible. Non negative results should only be interpreted at \_\_\_\_\_.
  - a. 10 minutes
  - b. 1 minute
  - c. 3 minutes
  - d. 5 minutes

**ANSWER:** \_\_\_\_\_

### II. MATCHING (Match the appropriate abbreviation to the drug name)

- |                     |       |      |
|---------------------|-------|------|
| 1. Marijuana        | _____ | COC  |
| 2. Amphetamines     | _____ | PCP  |
| 3. Cocaine          | _____ | mAMP |
| 4. Opiates/Morphine | _____ | AMP  |
| 5. Phencyclidine    | _____ | OPI  |
| 6. Methamphetamine  | _____ | THC  |

**Please fill out your contact information to receive a Certificate of Completion:**

Name \_\_\_\_\_




Company \_\_\_\_\_

Address \_\_\_\_\_

\_\_\_\_\_

Phone (\_\_\_\_\_) \_\_\_\_\_

### III. IDENTIFY THE RESULTS AS EITHER: NON NEGATIVE, NEGATIVE, OR INVALID

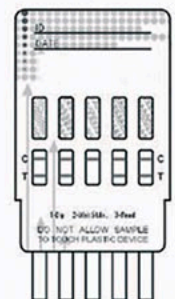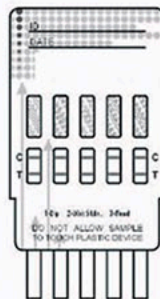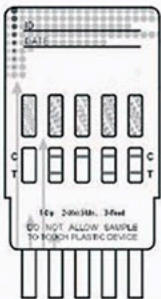

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_